Clarity C320

Corded Telephone with Answering Machine
Featuring Clarity® Power™ Technology

User Guide

E
N
G
L
I
S
H

E
S
P
A
Ñ
O
L

F
R
A
N
Ç
A
I
S

CLARITY 
Amplifying Your Life™

Table of Contents

SAFETY INSTRUCTIONS	3
UNPACKING THE C320	6
INSTALLATION	7
FEATURES	11
• Clarity Power Control™	12
• Boost Control	12
• Clarity Effect	12
• Boost Reset Switch	12
• Ringer Controls	13
• Visual Ringer	13
• Tone / Pulse Selection	13
• Last Number Redial	14
• Flash	14
• Hold	14
• Headset Port	15
• Backlite Keypad	15
• Nite Lite	15
• Neck Loop/Auxillary Port	16
OPERATION	16
• To Place or Receive a Call	16
• Programming Frequently-Dialed Numbers	17
• Answering System	18
CARE	20
• Trouble-shooting Guide	21
• FCC Requirements	24
• Warranty and Service	26

IMPORTANT SAFETY INSTRUCTIONS

When using your telephone equipment, basic safety precautions should always be followed to reduce the risk of fire, electric shock and persons including the following:

1. Read and understand all instructions.
2. Follow all warnings and instructions marked on the telephone.
3. Do not use this telephone near a bathtub, wash basin, kitchen sink or laundry tub, in a wet basement, near a swimming pool or anywhere else there is water.
4. Avoid using a telephone (other than a cordless type) during a storm. There may be a remote risk of electrical shock from lightning.
5. Do not use the telephone to report a gas leak in the vicinity of the leak.
6. Unplug this telephone from the wall outlets before cleaning. Do not use liquid cleaners or aerosol cleaners on the telephone. Use a damp cloth for cleaning.
7. Place this telephone on a stable surface. Serious damage and/or injury may result if the telephone falls.
8. Do not cover the slots and openings on this telephone. This telephone should never be placed near or over a radiator or heat register. This telephone should not be placed in a built-in installation unless proper ventilation is provided.
9. Operate this telephone using the electrical voltage as stated on the base unit or the owner's manual. If you are not sure of the voltage in your home, consult your dealer or local power company.
10. Do not place anything on the power cord. Install the telephone where no one will step or trip on the cord.
11. Do not overload wall outlets or extension cords as this can increase the risk of fire or electrical shock.
12. Never push any objects through the slots in the telephone. They can touch dangerous voltage points or short out parts that could result in a risk of fire or electrical shock. Never spill liquid of any kind on the telephone.
13. To reduce the risk of electrical shock, do not take this phone apart. Opening or removing covers may expose you to dangerous voltages or other risks. Incorrect reassembly can cause electric shock when the appliance is subsequently used.

SAVE THESE INSTRUCTIONS

IMPORTANT SAFETY INSTRUCTIONS

14. Unplug this product from the wall outlets and refer servicing to the manufacturer under the following conditions:
 - A. When the power supply cord or plug is frayed or damaged.
 - B. If liquid has been spilled into the product.
 - C. If the telephone has been exposed to rain or water.
 - D. If the telephone does not operate normally by following the operating instructions. Adjust only those controls that are covered by the operating instructions. Improper adjustment may require extensive work by a qualified technician to restore the telephone to normal operation.
 - E. If the telephone has been dropped or the case has been damaged.
 - F. If the telephone exhibits a distinct change in performance.
15. Never install telephone wiring during a lightning storm.
16. Never install telephone jacks in wet locations unless the jack is specifically designed for wet locations.
17. Never touch uninsulated telephone wires or terminals unless the telephone line has been disconnected at the network interface.
18. Use caution when installing or modifying telephone lines.
19. Use only the power cord and batteries indicated in this manual. Do not dispose of batteries in a fire. They may explode. Check with local codes for possible special disposal instructions.

Important Safety Instructions

The following items are included as part of the CS-03 Requirements. The standard connecting arrangement for the equipment is CA11A. This product meets the applicable Industry Canada technical specifications.

NOTICE: The Canadian Department of Communications label identifies certified equipment. This certification means that the equipment meets certain telecommunications network protective operational and safety requirements. The Department does not guarantee that the equipment will operate to the user's satisfaction.

SAVE THESE INSTRUCTIONS

SAFETY INSTRUCTIONS

Before installing this equipment, users should ensure that it is permissible to be connected to the facilities of the local telecommunications company. The equipment must also be installed using an acceptable method of connection. In some cases, the company's inside wiring associated with a single line individual service may be extended by means of a certified connector assembly (telephone extension cord). The customer should be aware that compliance with the above conditions may not prevent degradation of service in some situations. Repairs to certified equipment should be made by an authorized Canadian maintenance facility designated by the supplier. Any repairs or alterations made by the user to this equipment, or equipment malfunctions, may give the telecommunications company cause to request the user

disconnect the equipment. Users should ensure for their own protection that the electrical ground connections of the power utility, telephone lines and internal metallic water pipe system, if present, are connected together. This precaution may be particularly important in rural areas. Caution: Users should not attempt to make such connections themselves, but should contact the appropriate electric inspection authority, or electrician, as appropriate. The Ringer Equivalent Number is an indication of the maximum number of terminals allowed to be connected to a telephone interface. The termination on an interface may consist of any combination of devices subject only to the requirement that the sum of the Ringer Equivalent Number of all the devices not exceed five.

SAVE THESE INSTRUCTIONS

UNPACKING THE C320


Inside you will find the following items:

- A. Handset
- B. Handset Line Cord (Coiled)
- C. Base Unit
- D. Phone Line Cord (long, flat)
- E. Mounting Bracket
- F. AC Adapter
- G. User's Guide

equipment. Please call 1-800-552-3368 to exchange this unit for a factory-fresh replacement. In addition to the items listed above, you will need one of the following to complete installation of this unit:

- Modular Wall Jack, or
- RJ-11C Modular Jack Adapter (if you have an older 4-prong jack)


Carefully remove all of the contents from the package. If there is any visible damage, or any of the contents are missing, do not attempt to operate this

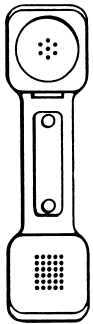
A


B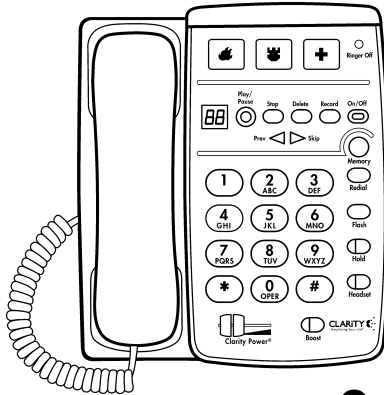


C

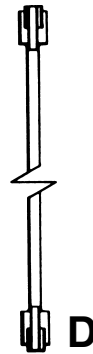
D

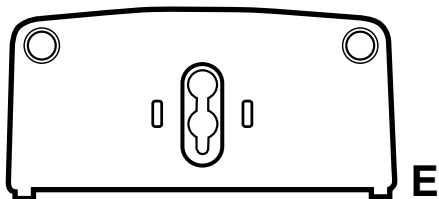E

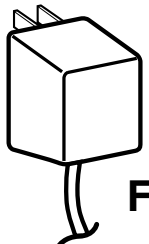F

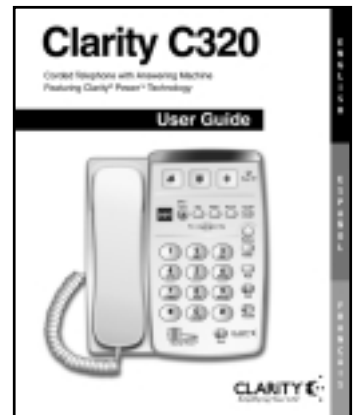G

INSTALLATION

Warning: To prevent electrical shock, always remove the telephone line cord from the wall jack before removing the cord from the telephone. Never leave the line cord connected to the wall jack if it is not also connected to the telephone.

AC adapter should be unplugged when not in use.

► **Select a Location:**

- This unit requires a modular phone jack and a standard 120 volt outlet.
- The unit should be located in an area where it is easy to see the lighted ringer.
- The unit can be used as a desk top phone or a wall mount phone.

► **Connecting Your C320**

The C320 can be used as a desk or wall telephone.

To install as a Desk Unit:


- Insert AC Adapter into the jack located on the top side of the telephone.

- Locate the hinge slots located on the right and left side of the back of the telephone.
- Turn wall mount bracket so the wide end (end with rubber feet) of the bracket is towards the top.
- Install bracket by inserting the tabs on the bracket into the hinge slots.
- Press down on the top of the bracket to lock into place.
- Plug AC Power Adapter directly into standard 120 volt outlet.
- Plug one end of the telephone line cord (flat) into the back of the telephone.
- Plug the other end of the telephone line cord into the wall jack.
- Plug one end of the handset cord (coiled) into the bottom of the handset.
- Plug other end of handset cord into the jack located on the lower left side of the telephone and hang up receiver.
- The unit should sit on an angle when used as a desk phone.

INSTALLATION


► Installation View

Note: If you are plugging the phone into an older 4-prong jack, use an RJ11C adapter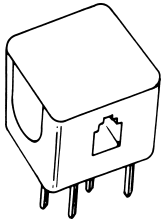

RJ-11C

Coiled cord plugs-in on this side of phone

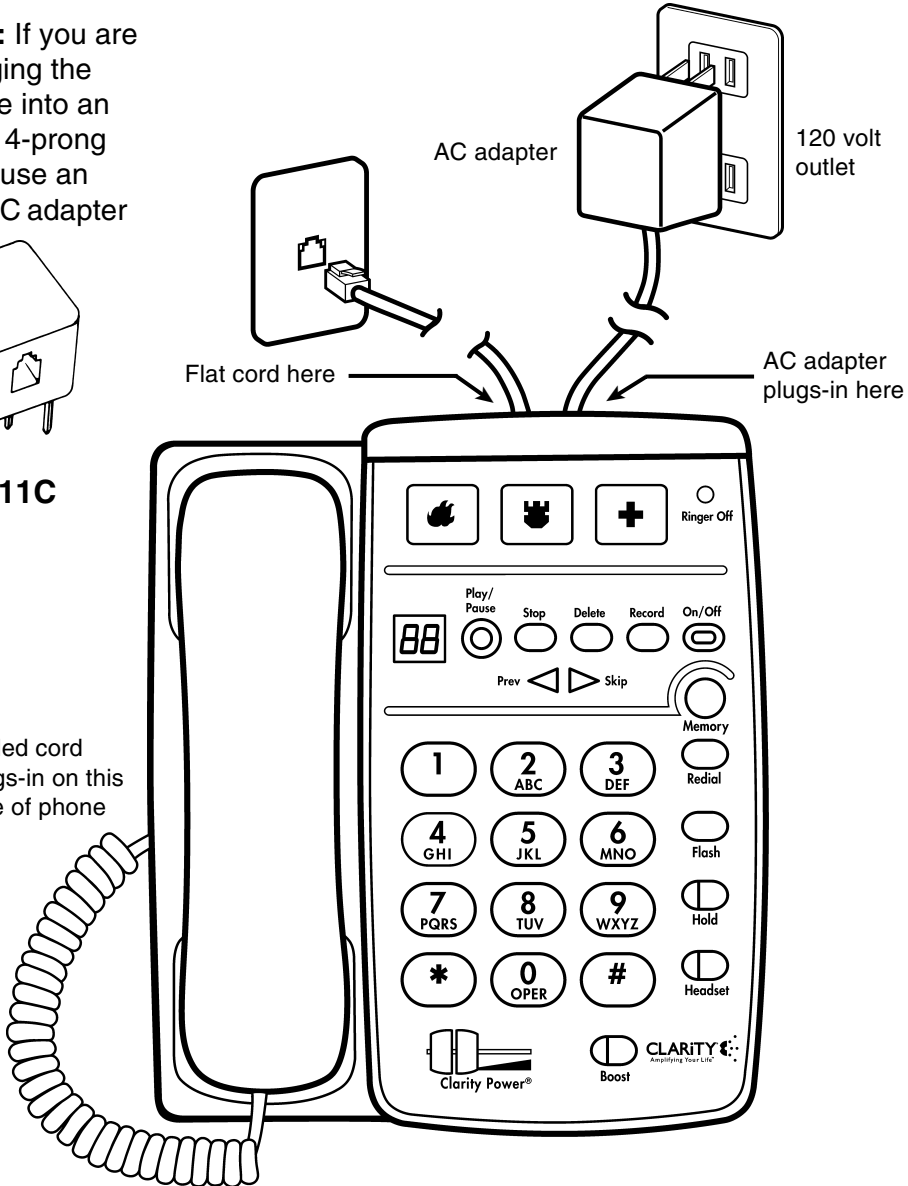INSTALLATION

To Install As a Wall Unit:

Important: A modular wall jack with raised mounting pins is required for installation as a Wall Unit.

- Insert AC Adapter into the jack located on the top side of the telephone.
- Locate the hinge slots located on the right and left side of the back of the telephone.
- Insert AC Power cord into the straight groove located on the back of the telephone.
- Turn bracket so that the wide end (end with rubber feet) of the bracket is towards the bottom.
- Install bracket by inserting the tabs on the bracket into the hinged slots.
- Feed power cord through opening at the bottom of the bracket. The cord will run underneath the bracket.
- Press down on the bottom of the bracket to lock into place.
- Plug AC adapter directly into standard 120 volt outlet.
- Plug one end of the telephone line cord into the back of the C320 telephone.
- Plug the other end of the telephone line cord into the wall jack.

Back of phone

Mounting Bracket


INSTALLATION

- When wall mounting the telephone feed the line cord into the angled groove on the back of the telephone and plug into wall jack.
- Hold the C320 so that the holes on the mounting bracket line up with the raised mounting pins on the modular wall jack. Once the holes are lined up, push the phone in towards the wall jack and then slide the phone downward until it is securely seated on the wall jack.
- Plug one end of the handset cord (coiled) into the bottom of the handset.
- Plug the other end of the handset cord into the jack located on the lower left edge of the C320 telephone and hang up receiver.
- When the phone is wall mounted the handset knob will need to be reversed to hold the handset in place. To reverse, pull up on the knob and turn a quarter turn clockwise. The knob will hold the handset in place when you hang up.
- To convert phone from one mounting position to the other simply depress the tabs located above the feet on the mounting plate and lift bracket off base of the phone. Follow the instructions above for positioning the bracket.

Avoid placing the unit where it will be exposed to excessive heat or humidity.

C320 FEATURES

- | | | |
|-----------------------------|---------------------------------|--|
| 1. Clarity Power Control™ | 12. Answering System Stop | 21. Answering System Ring Selection |
| 2. Boost | 13. Answering System Play/Pause | 22. Headset Jack (2.5mm) |
| 3. Headset Button/Indicator | 14. New Call Display | 23. Key Pad and Memory Storage Locations |
| 4. Hold Button/Indicator | 15. Answering System Microphone | 24. Neckloop Jack (3.5mm) |
| 5. Flash | 16. Ringer Style | 25. AC Adapter Plug In |
| 6. Last Number Redial | 17. Ringer Volume Down | 26. Nite Light Off/Lo/Hi |
| 7. Memory Storage Button | 18. Ringer Volume Up | 27. Boost Reset |
| 8. Ringer On/Off Indicator | 19. Ringer Off/On | 28. Answering System Previous |
| 9. Answering System On/Off | 20. Tone/Pulse | 29. Answering System Skip |
| 10. Answering System Record | | 30. Emergency Storage Locations |
| 11. Answering System Delete | | |

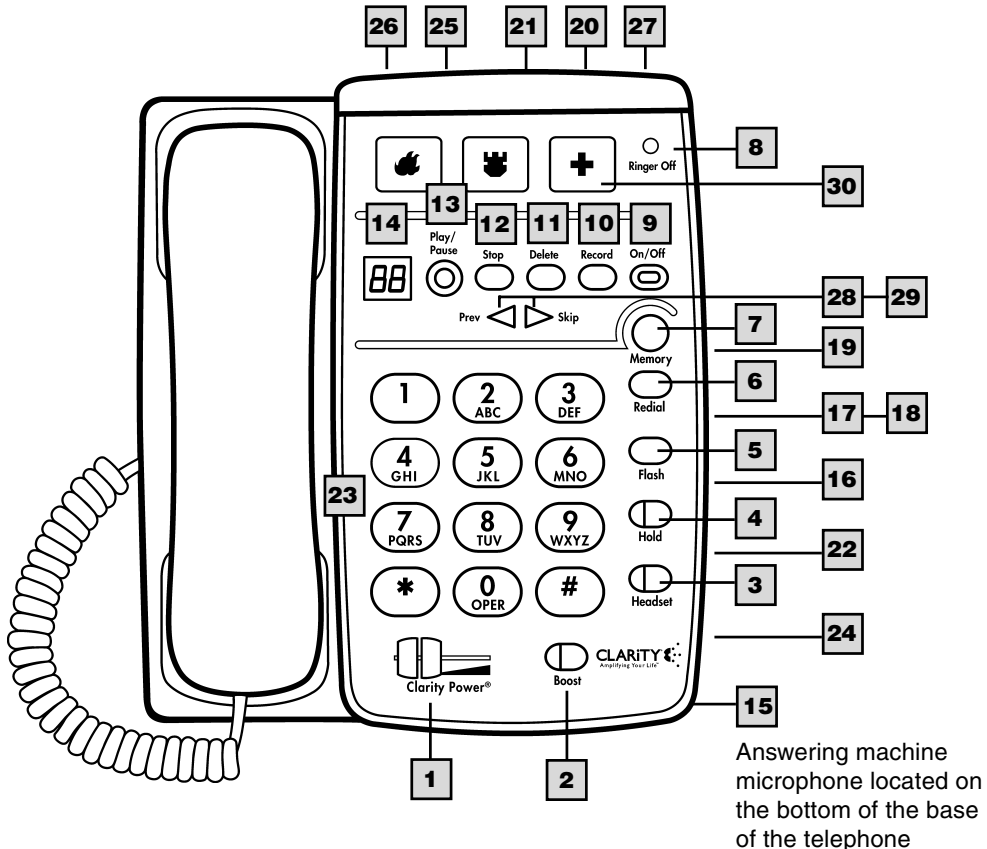C320 FEATURES

Your Clarity telephone is packed with features that will help you improve your communications over the telephone.

► **Clarity® Power™ Control**

The Clarity Power Control allows you to control the clarity of your phone conversations. It will let you hear clearly phone calls that are normally difficult to understand. The amplification will be increased up to 30 decibels and the frequency band from 2000 to 3000Hz will be specially enhanced to improve the clarity of your call.

► **Boost Control**

The C320 has a BOOST button that controls the loudness of the receiver. Once the BOOST button is pressed, the BOOST indicator will light and the volume will jump to the volume level indicated by the volume control.

► **To Increase Clarity Power**

Slide the button marked **Clarity Power** toward the right.

► **To Decrease Clarity Power**

Slide the **Clarity Power** button toward the left. The extreme left is the level of a normal telephone conversation.

► **Boost Reset Switch**

The C320 has a reset switch located on the bottom of the unit. This switch will disable the reset feature of the boost button upon hang up as follows:

ON—When the Boost Reset switch is in “on” position volume will remain at the current volume control setting. This eliminates the need to press the BOOST button with each phone call.

OFF—When the Boost Reset switch is in “off” position volume will reset to nominal (normal) upon hangup. The BOOST button will need to be depressed with each phone call to restore settings.

WARNING: When BOOST button is lit volume may be at a high level. To protect hearing, lower volume control to minimum.

C320 FEATURES


► Ringer Controls

The volume of ring can be adjusted by pressing the ringer volume up switch or down switch until the desired ring volume is achieved.

The style of ring can also be adjusted by pressing the Ringer Style button. Six ringer styles are available allowing you to customize your ring to your hearing needs.

You can check the volume and style of ring selected without waiting for an incoming call. The phone will ring indicating the style and volume selected by pressing the ringer style button.

When the Ringer ON/OFF is set to OFF, the red Ringer Off Indicator on the front of the phone will illuminate. The Visual Ringer will flash to alert you of an incoming call.

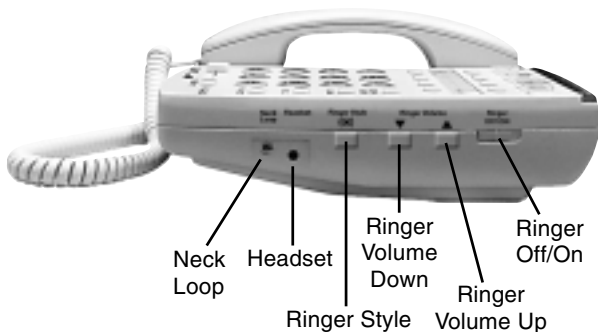► Visual Ringer

A lighted visual ringer works in conjunction with the low frequency ringer. It sends out a flashing alert every time a phone call is received. (The ringer will light up regardless of the setting on the ringer volume control.)

► Tone/Pulse Selection

The C320 phone can be used as either a TONE (touch dial) or PULSE (rotary dial) phone. Tone dialing is required for using special phone services such as long distance calling. Please check with your local phone company on certain types of service.

Note for Users with Pulse/Rotary service only: Many automated answering systems used by banks, stores and government agencies ask you to “Push 1 for service”, “Push 2 for the operator”, etc. You must switch the tone/pulse switch on the top edge of the phone to tone before responding to these prompts. In summary, dial the phone number you want to call in pulse mode, switch to tone mode to interact with the automated answering system, switch back to pulse mode when your call is done. When switching from Pulse to Tone during a call, there will be a 3 second delay from the time you first push a key in Tone mode until the key press is transmitted. You cannot switch from tone to pulse during a call.

C320 FEATURES

Important: You must have tone signaling service to use the tone feature on the C320. If in doubt, contact your local telephone company.

► To Set the TONE/PULSE Switch:

1. Move the switch towards the correct setting.
2. Depress the hookswitch once.
3. The phone will now dial in the mode you have selected.

► Last Number Redial

Last Number Redial allows you to automatically redial the last telephone number you called. This feature is convenient for redialing numbers which are busy. The phone number stored for redial will be reset with every new call you make.

► To Operate Last Number Redial:

1. Depress the hookswitch to get a new dial tone.
2. Press the **REDIAL** button once.
3. Stay on the line, and your call will be dialed.

The last number dialed will be stored in the C320 for you until you dial another number or disconnect the telephone from the wall jack.

► Flash

Flash is a special feature that is required to use some calling services provided by your local phone company, such as call waiting. Consult your guide to these services which is available from your local phone company. Flash serves the same purpose as pressing and releasing the hook switch very quickly.

► To Use the Flash Feature:

Simply press the **FLASH** button at the appropriate time as outlined in the guide provided by your local phone company.

► Hold

This advanced feature on your C320 allows you to put a call on hold, hang up and switch to another extension or have a private conversation that you don't want the person on the other end of the phone to hear.

C320 FEATURES

You will not be able to hear the person on the other end and they will not be able to hear you while the **Hold Indicator Light** is on. The hold light will be on any time the hold button has been pressed. A call will stay on hold until you pick up a receiver, or press and release the hook switch. Pressing the hold button a second time will not take the call off hold.

► To Use the Hold Feature:

1. While a telephone call is in progress, press the **HOLD** button once to place the call on hold. The red hold light will turn on.
2. Hang up the receiver.
3. To take the call off hold:
 - A. Simply lift the C320 receiver out of the cradle.
-OR-
 - B. Pick up another telephone extension in your home.
-OR-
 - C. If you did not return the handset to the cradle after pushing the hold button, press and release the hook switch once only.
4. The hold light will turn off.

► Headset Port

The C320 is equipped with a 2.5 mm jack for use with a compatible headset, allowing for use in a hands free mode. To attach headset, plug the headset into the headset port located on the right side of the telephone. Depressing the headset button on the front allows calls to be made without lifting the handset to seize the line. When the headset is plugged in, the hand-set is disabled.

► Backlite Keypad

The keypad features large translucent backlite buttons that will make calling easier.

► Nite Lite

The Nite Lite feature illuminates the key pad. The Nite Lite makes the keypad visible when the phone is hung up, allowing it to be easily located at night. The Nite Lite can be adjusted to OFF, LO and HIGH.

C320 FEATURES

► Neck Loop/Auxillary Port

The C320 has a 3.5mm phono jack to use with a neck loop. The volume to the neck loop is controlled with the Clarity Power Control. When the neck loop is plugged in, the receiver in the handset is disabled.

► Operations

Once you have installed your Walker C320 according to the instructions noted previously, it will be ready to use.

► To Place or Receive a Call:

1. Adjust the Clarity® Power Control™ to its lowest setting (zero).
2. Pick up the handset and listen for the caller's voice or a dial tone.
3. When placing a call, dial as you would any phone and wait for someone to answer.

4. Press the **BOOST** button.

5. Adjust the Clarity Power Control upwards to a level where the other party's voice is heard loud and clear, and begin conversation.

6. When you complete the call, return the handset.

7. After hanging up, the sound level will reset according to the position of the reset switch on the back of the telephone. (See Boost Reset Switch instructions on page 12.)

8. Pressing the BOOST Button during subsequent conversations will return the volume to your set sound level.

Note: If you made an error while dialing, just depress the hookswitch, listen for a dial tone, and dial again.

PROGRAMMING NUMBERS INTO MEMORY

► Memory Storage Locations

The C320 allows you to program ten (10) indirect or two touch memory locations on the telephone (keys 0-9) that can be programmed to dial frequently called numbers of up to 16 digits. You can also program numbers into the three (3) extra large memory storage locations for one touch dialing.

Preprinted inserts have been provided for you to place in the extra large programmable buttons. To remove the button covers, locate the ridge on the right and left side of the button. Grasp the ridge with your finger tips and pull up on the button, removing it from the telephone. Insert the selected symbol into button cover and place back into the telephone.

► Storing Numbers Into Memory

1. Press the **MEMORY** button.
2. Enter the phone number you wish to store to memory.
3. Press the **MEMORY** button again.
4. Press either the emergency storage location or the number on the keypad that you wish to save the number to.
5. The **HOLD** light will flash twice to indicate memory storage is complete.

► Dialing From Memory

1. Pick up the handset.
2. Press the **MEMORY** button.
3. Press the number on the keypad where the number to be dialed has been stored.
4. The call will dial automatically.
5. To dial the emergency storage locations simply press the desired button. The call will dial automatically.

► Editing Memory Numbers

Replace a number stored into memory by programming a new number in its place.

► Deleting Numbers Stored in Memory

1. Press and release the **MEMORY** button.
2. Press the **MEMORY** button again.
3. Press the number on the keypad you wish to delete the number from.
4. The **HOLD** light will flash twice to indicate the process is complete.

ANSWERING SYSTEM OPERATION

Answering System

IMPORTANT: You must record an outgoing message in order for the answering system to function. The display will flash OR indicating outgoing message needs to be recorded.

The C320 answering system will be ON when you first supply power to it, and the ON/OFF button will be lit.

Repeated presses of the ON/OFF button will turn the answering system on or off. If the answering system is off, the ON/OFF button will not be lit and no messages will be recorded.

Each incoming message is limited to a maximum length of sixty (60) seconds. A warning tone will sound ten (10) seconds before the system stops recording.

The total recording time for all messages is fourteen (14) minutes.

Outgoing Message

A personal outgoing message must first be recorded in order for the answering system to operate. The maximum length of the outgoing message is one (1) minute.

Recording Outgoing Message

1. Press and hold the **RECORD** button for (two) 2 seconds.

2. After the beep, begin recording. Speak clearly into the MIC in the base (not handset) to record your outgoing announcement. The record time of the outgoing message will be displayed while recording.

3. When finished, press **STOP**.

4. The outgoing message will automatically play. While the outgoing message is playing, the display will flash **OP**. You can press the **STOP** button again to skip or wait until message has played.

To Change Your Greeting

To clear your greeting, record but do not speak during the recording. The display will indicate elapsed record time. Press **STOP** button to finish. The display will flash **OR** while recording.

Checking Outgoing Message

You can listen to the outgoing message at any time by pressing the **RECORD** button once. **OP** will flash during playback.

Number of Rings

You can select the number of times the C320 will ring before the answering system answers a call. You may select from 2, 4 or 6 rings. To set, slide the switch located on the top end of the phone to the desired setting.

ANSWERING SYSTEM OPERATION

Monitoring Incoming Calls

You are able to hear messages as your callers leave them through the speaker in the base. Lifting the handset while a caller is leaving a message will stop recording and allow you to talk.

Listening to Messages

When a new message is received, the light on the **PLAY** button will flash Red and the total number of incoming messages will be displayed.

Use the **PLAY** and **STOP** buttons to start and stop message playback. Pressing **PLAY** will play new messages first. Use the **PREV** button for the olds. If there is no new message in the machine, it plays older messages starting from the first record. The display will show the number of the message record.

Playback can **ONLY** start while the handset is in the cradle. The light on the **PLAY** button will shine Green while messages are playing.

To listen to messages with Clarity Power, lift the handset during message playback. The volume and clarity of the message can then be controlled by the Clarity Power slide control.

During Playback

To Pause a Message

Press the **PLAY/PAUSE** button to pause a message while playing. The **PLAY** button will flash Green while message playback is paused. Pressing **PLAY** will resume message playback. During pause, the number of the message record will be displayed.

To Repeat a Message

Press the **PREV** button to return to the beginning of the current message. Press the **PREV** button twice to return to the previous message. The number of the message record will be displayed.

To Skip a Message

Press the **SKIP** button to advance ahead to the next message. The number of the message record will be displayed.

Receiving New Calls During Message Playback

The answering system will stop message playback if a phone call is received during message playback.

ANSWERING SYSTEM OPERATION

Handset Mode

If a new phone call is received during message playback in the handset mode (handset off hook), message playback will continue. The visual ringer will flash to indicate the incoming call but the phone will not ring. To answer the phone call, press the answering systems on/off switch or depress the hook switch once. To have the answering system answer the phone call place the handset in the cradle.

Speaker Phone Mode

If a new phone call is received during the message playback while in the speaker-phone mode (handset on hook), message playback will stop and you will be able to

monitor the outgoing message announcement and the new incoming message. To speak to the caller, lift the handset.

Deleting Messages

To delete a specific message press the **DELETE** button while the message is playing. The total number of messages will be updated after playback.

To delete all messages press and hold the **DELETE** button for (three) 3 seconds. Messages can only be deleted while the phone and answering system are not being used. The display will indicate "00" messages are stored.

CARE

The C320 has been designed to give you years of dependable service with a minimum of care.

To ensure the utmost in performance, follow these guidelines in caring for the unit:

- Avoid dropping or knocking the unit.
- Dust the unit regularly. Wipe the unit with a soft, damp cloth. Do not use detergents and avoid excess moisture.
- The C320 is an electrical device. Avoid electric shock by keeping the phone away from water. (i.e.: bathroom, kitchen sink, etc.)

TROUBLE-SHOOTING GUIDE

► Phone Will Not Operate

Disconnect any other equipment that may be attached to the phone. Check your phone cords and connections. Try a different line cord and/or handset cord to determine if the problem is with one of the cords. If problem lies in the cord simply replace the cord.

► Dialing Problems

• Can't Dial Out

Make sure that TONE/PULSE switch is set correctly. Phone should be set to TONE if you have touch tone service from your phone company. Phone should be set to PULSE if operating with rotary dial service.

• Phone Doesn't Dial Out On First Attempt

Your phone may be at the end of a long line of phones, due to the way the phone company installed the wiring in your neighborhood. In this case, your phone may not get enough power from the phone line. Try waiting 3-5 seconds after lifting the handset before dialing the number.

• Loss Of Numbers Programmed Into Memory

Your C320 phone will maintain its memory for up to 12 hours during a power loss. If you unplug the phone from the telephone line for long periods or lose your phone service for an extended period of time you will need to reprogram the phone.

► Programming Personal Access Codes

• Difficulty Programming Account Numbers

Your TONE/PULSE selector must be switched to the TONE mode to enter account numbers. This is because most interactive phone systems require TOUCH TONE to operate. If your phone service is rotary service, you must have the switch set to PULSE to dial a number. Once you are connected move the switch to TONE and enter your responses using TOUCH TONE mode. When your call is complete move the switch back to PULSE so you are ready to make your next call.

TROUBLE-SHOOTING GUIDE

► **Static On Line**

• **Interference From Household Appliances**

Your C320 phone may be located near a touch lamp, microwave or refrigerator which may cause interference. Relocating your phone to another area may eliminate the problem.

A cordless phone on your phone line may also cause static. Try disconnecting your cordless phone for several days. This may eliminate the problem.

Disconnect any extra devices which may be attached to the phone. Extra devices may include cord detangler's, caller ID units, and answering machines.

• **Interference From Malfunctioning, Frayed Or Poorly Connected Telephone Cords**

Check your phone cords and connections. Try interchanging cords with another phone to see if the problem is eliminated. Replace any defective equipment.

► **Feedback Problems.**

• **Feedback Occurs When Handset Is Set Down**

Avoid setting receiver on a hard surface or attach a foam receiver pad to earpiece of phone. You can also lay the receiver on its side. This will eliminate feedback also.

► **Transmission Problems**

• **The Person On The Other End Cannot Hear You**

Too many phones or phone devices on the line may effect your phone's transmission. As a guideline, more than five (5) phones or phone devices may overload the phone line. Disconnect one (1) or two (2) of the devices to see if that eliminates the problem. If not, contact your local phone company for load guidelines.

You may not be speaking directly into the transmitter. Always speak directly into the mouthpiece in your normal tone of voice.

TROUBLE-SHOOTING GUIDE

► **Compatibility**

• **The Clarity 320 Is Compatible With Digital PBX Systems**

If a standard, single line phone works, your C320 phone should also work.

• **The C320 Phone is Not Compatible With Key Systems Or AT&T PBX Systems**

These units have a large electronic box controlling many phones as one unit. These types of units may be found in office buildings and some retirement homes, nursing homes and condos.

► **Ringer Volume Problem**

• **Unable To Hear Phone Ringer**

Make sure the ringer volume is not set to ON. Increases the volume of the ring if necessary.

Too many phone or phone devices on a line may cause your phone to not ring. Disconnect one (1) or (2) two of the devices to see if the problem is eliminated. If not, contact your local phone company for load guidelines.

Consider purchasing a variable pitch auxiliary ringer such as the Clarity WR100. These devices are exceptionally loud and can be plugged into any phone outlet in your home.

► **Interference Problems**

• **User Hears Radio Or CB Transmissions Over The Phone**

Contact your local phone company to have your telephone line checked. A Radio Frequency Interference (RFI) filter may be put on the line to minimize this problem.

If you have any telephone service problems, first make sure you have followed all of the installation procedures outlined in this user's guide. Then try to determine if the source of the problem is the C320 itself or your phone service. To do this, disconnect the C320 and plug in your standard telephone. If the trouble persists with the standard telephone, inform your telephone company. If the trouble is related to only the C320, review the installation procedures and the troubleshooting guide before referring to the Limited Warranty stated later in this user guide.

FEDERAL COMMUNICATIONS COMMISSIONS REQUIREMENTS

FCC Part 68 Information

This equipment complies with Part 68 of FCC Rules and the requirements adopted by the ACTA. The FCC Part 68 Label is located on the BASE UNIT of the telephone. This label contains, among other information, a product identifier in the format US:AAAEQ##TXXXX. If requested, this information must be provided to your telephone company.

Connection to the network should be made using standard modular telephone jacks, USOC type RJ11C. The plug and/or jacks used must comply with FCC Part 68 Rules and requirements adopted by the ACTA. A compliant telephone cord and modular plug is provided with this product. It is designed to be connected to a compatible modular jack that is also compliant. See installation instructions for details.

The REN is useful in determining the quantity of devices that you may connect to your telephone line and still allow these devices to ring when your telephone number is called. Excessive RENS on a telephone line may result in the devices not ringing in response to an incoming call. In most areas, but not all, the sum of the RENs for all devices that you may connect to your line should not exceed five (5.0). To be certain of the number of devices that you may connect to your line, you should contact your local telephone company. The REN for this product is part of the product identifier that has the format US: AAAEQ##TXXXX. The digits

represented by ## are the REN without a decimal point.

If this telephone equipment, the Clarity 320 telephone, causes harm to the telephone network, the telephone company will notify you in advance that temporary discontinuance of service may be required. But, if advance notice isn't practical, the telephone company will notify the customer as soon as possible. Also, you will be advised of your right to file a complaint with the FCC if you believe it is necessary.

The telephone company may make changes in its facilities, equipment, operations or procedures that could affect the proper functioning of your equipment if they do, you will be notified in advance in order for you to make the necessary modifications to maintain uninterrupted service.

If trouble is experienced with this unit, for repair or warranty information, please contact customer service at the phone number listed in the Warranty section. If the equipment is causing harm to the network, the telephone company may request that you disconnect the equipment until the problem is resolved.

DO NOT DISASSEMBLE THIS EQUIPMENT. It does not contain any user serviceable components.

Connection to party lines is subject to tariffs. Contact the state public utility commission, public service commission or corporation commission for information.

FEDERAL COMMUNICATIONS COMMISSIONS REQUIREMENTS

If your home has specially wired alarm equipment connected to the telephone line, please ensure that the installation of this equipment does not disable alarm equipment by consulting your telephone company or a qualified installer.

This equipment is hearing aid compatible.

We recommend the installation of an AC surge arrester in the AC outlet to which this equipment is connected. The telephone companies report that electrical surges, typically lighting transients, are very destructive to customer terminal equipment connected to AC power sources.

Part 15 of FCC Rules Interference Information

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Your Clarity 320 has been tested and found to comply with the limits of a Class B digital device, pursuant to Part 15 of FCC rules. These limits are designed to provide reasonable protection against harmful interference in residential installation.

This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However,

there is no guarantee that interference will not occur in a particular installation; if this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, you are encouraged to try to correct the interference by one of the following measures:

1. Where it can be done safely, reorient the receiving television or radio antenna.
2. To the extent possible, relocate the television, radio or other receiver with respect to the telephone equipment. (This increases the separation between the telephone equipment and the receiver.)
3. Connect the telephone equipment into an outlet on a circuit different from that to which the television, radio, or other receiver is connected.
4. Consult the dealer or an experienced radio/TV technician for help.

CAUTION: Changes or modifications not expressly approved by the manufacturer responsible for compliance could void the user's authority to operate the equipment.

THE PARTY RESPONSIBLE FOR PRODUCT COMPLIANCE

Clarity, A Division of Plantronics, Inc.
4289 Bonny Oaks Drive, Suite 106
Chattanooga, TN 37406
Telephone: 1-800-552-3368

WARRANTY AND SERVICE

The following warranty and service information applies only to products purchased and used in the U.S. and Canada. For warranty information in other countries, please contact your local retailer or distributor.

Limited Warranty

Clarity, a division of Plantronics, Inc. (“Clarity”) warrants to the original consumer purchaser that, except for limitations and exclusions set forth below, this product shall be free from defects in materials and workmanship for a period of one (1) year from the date of original purchase (“Warranty Period”). The obligation of Clarity under this warranty shall be limited to repair or replacement, at Clarity’s option, without charge, of any part or unit that proves to be defective in material or workmanship during the Warranty Period.

Exclusions from Warranty

This warranty applies only to defects in factory materials and factory workmanship. Any condition caused by accident, abuse, misuse or improper operation, violation of instructions furnished by Clarity, destruction or alteration, improper electrical voltages or currents, or repair or

maintenance attempted by anyone other than Clarity or an authorized service center, is not a defect covered by this warranty. Telephone companies manufacture different types of equipment and Clarity does not warrant that its equipment is compatible with the equipment of a particular phone company.

Implied Warranties

Under state law, you may be entitled to the benefit of certain implied warranties. These implied warranties will continue in force only during the warranty period. Some states do allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

Incidental or Consequential Damages

Neither Clarity nor your retail dealer or selling distributors has any responsibility for any incidental or consequential damages including without limitation, commercial loss or profit, or for any incidental expenses, loss of time, or inconvenience. Some states do not allow exclusion or limitation of incidental or consequential damage, so the above limitation or exclusion may not apply to you.

WARRANTY AND SERVICE

Other Legal Rights

This warranty gives you specific legal rights and you may also have other rights which vary from state to state.

How to Obtain Warranty Service

To obtain warranty service, please prepay shipment and return the unit to the appropriate facility listed below.

In the United States:

Clarity Service Center
4289 Bonny Oaks Drive, Suite 106
Chattanooga, Tennessee 37406

Tel. (423) 622-7793 or
(800) 426-3738
Fax: (423) 622-7646 or
(800) 325-8871

In Canada:

Plantronics Service Center
1455 Pitfield Boulevard
Saint-Laurent
Quebec H4S 1G3

Tel: (800) 540-8363
(514) 956-8363
Fax: (514) 956-1825

Please use the original container, or pack the unit(s) in a sturdy carton with sufficient packing material to prevent damage. Include the following information:

1. A proof-of-purchase indicating model number and date of purchase
2. Bill-to address
3. Ship-to address
4. Number and description of units shipped
5. Name and telephone number of person to call, should contact be necessary
6. Reason for return and description of the problem

Damage occurring during shipment is deemed the responsibility of the carrier, and claims should be made directly with the carrier.

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©2004 Clarity, a Division of Plantronics, Inc. All rights reserved.
Control and Walker are trademarks or registered trademarks of Plantronics, Inc.