

CE30™ AMPLIFIED NECKLOOP

Important Safety Instructions

When using your telephone equipment, basic safety precautions should always be followed to reduce the risk of fire, electric shock and injury to persons, including the following:

- Read and understand all instructions.
- Follow all warnings and instructions marked on the product.
- Unplug this product from the wall outlet before cleaning. Do not use liquid cleaners or aerosol cleaners. Use a damp cloth for cleaning.
- Do not use this product near water, for example, near a bathtub, washbowl, kitchen sink, or laundry tub, in a wet basement, or near a swimming pool.
- Avoid using a telephone (other than a cordless type) during an electrical storm. There may be a remote risk of electric shock from lightning.
- Do not use this telephone to report a gas leak in the vicinity of the leak.
- Use only the power cord and batteries indicated in this manual. Do not dispose of batteries in a fire. They may explode. Check with local codes for possible special disposal instructions.


Safety Instructions for Batteries

Caution: Risk of explosion if battery is replaced by an incorrect type. Dispose of used batteries according to the instructions.

1. Use only two (2) AAA alkaline batteries in your neckloop.
2. Do not dispose of the battery in a fire as it may explode. Check with local codes for possible special disposal instructions.
3. Do not open or mutilate the battery. Released electrolyte is corrosive and may cause damage to the eyes and skin. It may be toxic if swallowed.
4. Exercise care in handling batteries in order not to short the battery with conducting materials such as rings, bracelets and keys. The battery or conduction material may over-heat and cause burns.

Contents and Parts Checklist

Neckloop	3.5mm adapter
(2) AAA batteries	User Guide
Battery Door	Screw

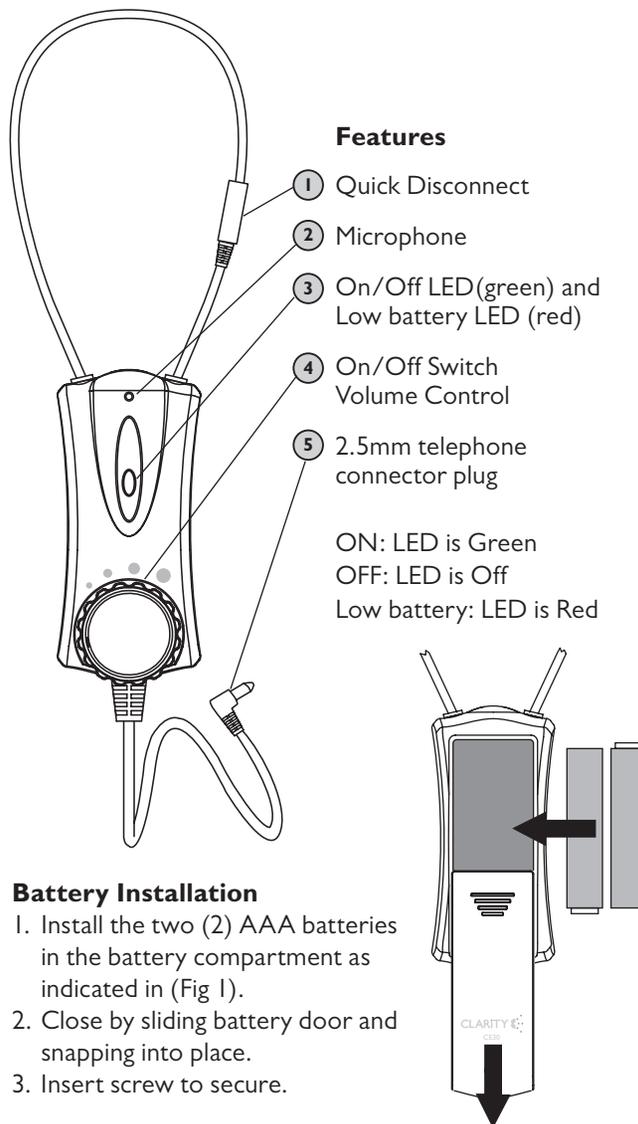

Features

- 1 Quick Disconnect
- 2 Microphone
- 3 On/Off LED (green) and Low battery LED (red)
- 4 On/Off Switch Volume Control
- 5 2.5mm telephone connector plug

ON: LED is Green
OFF: LED is Off
Low battery: LED is Red

Battery Installation

1. Install the two (2) AAA batteries in the battery compartment as indicated in (Fig 1).
2. Close by sliding battery door and snapping into place.
3. Insert screw to secure.

Battery Life

The battery life will depend upon usage and volume levels. The life of the batteries should last up to 30 to 40 hours before needing replaced.

Volume Control

The CE30 provides up to 30 dB of amplification. The volume is controlled by the power/volume dial located on top of the neckloop.

To Make or Receive a Call

1. Place neckloop around neck by connecting the quick disconnect. Be sure that the quick disconnect is closed securely in order for the CE30 to work properly.
2. Connect the 2.5mm plug into a telephone's 2.5mm headset jack.
3. Turn your hearing aid to "MT" or "T" mode.
4. While unit is around neck, turn the Power/Volume dial to the left for power to turn ON. The green LED will appear.
5. Make or Receive a call using the telephone. (For your telephone's operation, please see its user guide)
6. The CE30 features a built-in microphone, allowing for hands-free communication when plugged into a 2.5mm headset jack.
7. Adjust the volume to desired level using the dial on top of unit. **Note:** The volume on the telephone may also need to be adjusted.
8. When call is complete, turn dial off and unplug the adapter from telephone.
9. Please reset the hearing aid if needed.

Important: If your telephone does not have a 2.5mm jack an adapter may be needed. Check with your telephone's manufacturer.

Adapter

The CE30 comes with a 3.5mm adapter. This allows the user to use the neckloop with a 3.5 audio equipped telephone, CD player, MP3 player or any other device that contains an 8-16 Ohm audio output through a 3.5mm jack.

FCC REGISTRATION INFORMATION

1. This equipment complies with Part 68 of the FCC Rules. On the back of this equipment is a label that contains, among other information, the FCC registration number and Ringer Equivalence Number (REN) for this equipment. If requested, provide this information to your telephone company.
2. The registration jack USOC for the equipment is RJ11C.
3. An FCC compliant telephone cord and modular plug is provided with this equipment. This equipment is designed to be connected to the telephone network or premises wiring using a compatible modular jack which is Part 68 compliant. See installation instructions for details.
4. The REN is useful to determine the quantity of devices that may be connected to the telephone line. Excessive RENs on the telephone line may result in the devices not ringing in response to an incoming call. In most, but not all areas, the sum of RENs of all devices should not

exceed five (5). To be certain of the number of devices that may be connected to a line, as determined by the total RENs, contact the local telephone company.

5. If your telephone equipment causes harm to the telephone network, the telephone company will notify you in advance that temporary discontinuance of service may be required. But if advance notice isn't practical, you will be notified as soon as possible. You will be advised of your right to file a complaint with the FCC if you believe it is necessary.
6. Your telephone company may make changes in its facilities, equipment, operations, or procedures that could affect the operation of your equipment. If they do, you will be given advance notice so as to give you an opportunity to maintain uninterrupted service.
7. If you experience trouble with this equipment, please contact Clarity Customer Service Department at 1-800-426-3738 for repair/warranty information. If your equipment is causing harm to the telephone network, the telephone company may request that you disconnect the equipment until the problem is resolved.
8. This equipment may not be used on public coin service provided by the telephone company. Connection to party lines is subject to state tariffs. (Contact your state public utility commission or corporation commission for information.) **This equipment complies with Industry Canada CS03.**

The Load Number of the equipment is 20. The standard connecting arrangement code for the equipment is CA11A. **NOTICE:** The Canadian Department of Communications label identifies certified equipment. This certification means that the equipment meets certain telecommunications network protective operational and safety requirements. The Department does not guarantee that the equipment will operate to the user's satisfaction.

Before installing this equipment, users should ensure that it is permissible to be connected to the facilities of the local telecommunications company. The equipment must also be installed using an acceptable method of connection. In some cases, the company's inside wiring associated with a single line individual service may be extended by means of a certified connector assembly (telephone extension cord). The customer should be aware that compliance with the above conditions may not prevent degradation of service in some situations.

Repairs to certified equipment should be made by an authorized Canadian maintenance facility designated by the

supplier. Any repairs or alterations made by the user to this equipment, or equipment malfunctions, may give the telecommunications company cause to request the user disconnect the equipment. Users should ensure for their own protection that the electrical ground connections of the power utility, telephone lines and internal metallic water pipe system, if present, are connected together. This precaution may be particularly important in rural areas.

Caution: Users should not attempt to make such connections themselves, but should contact the appropriate electric inspection authority, or electrician, as appropriate. The Load Number (LN) assigned to each terminal device denotes the percentage of total load to be connected to a telephone loop which is used by the device to prevent overloading. The termination on a loop may consist of any combination of devices subject only to the requirement that the total of the load numbers of all the devices does not exceed 100.

WARRANTY & SERVICE

The following warranty and service information applies only to products purchased and used in the U.S. and Canada. For warranty information In other countries, please contact your local retailer or distributor.

Limited Warranty

Clarity, a division of Plantronics, Inc. ("Clarity") warrants to the original consumer Purchaser that, except for limitations and exclusions set forth below, this product shall be free from defects in materials and workmanship for a period of one (1) year from the date of original purchase ("Warranty Period"). The obligation of Clarity under this warranty shall be limited to repair or replacement, at Clarity's option, without charge, of any part or unit that proves to be defective in material or workmanship during the Warranty Period.

Exclusions from Warranty

This warranty applies only to defects in factory materials and factory workmanship. Any condition caused by accident, abuse, misuse or improper operation, violation of instructions furnished by Clarity, destruction or alteration, improper electrical voltages or currents, or repair or maintenance attempted by anyone other than Clarity or an authorized service center, is not a defect covered by this warranty. Telephone companies manufacture different types of equipment and Clarity does not warrant that its equipment is compatible with the equipment of a particular phone company.

Implied Warranties

Under state law, you may be entitled to the benefit of certain implied warranties. These implied warranties will continue in force only during the warranty period. Some states do allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

Incidental or Consequential Damages

Neither Clarity nor your retail dealer or selling distributors has any responsibility for any incidental or consequential damages including without limitation, commercial loss or profit, or for any incidental expenses, expenses, loss of time, or inconvenience. Some states do not allow exclusion or limitation of incidental or consequential damage, so the above limitation or exclusion may not apply to you.

Other Legal Rights

This warranty gives you specific legal rights and you may also have other rights which vary from state to state.

How to Obtain Warranty Service

To obtain warranty service, please prepay shipment and return the unit to the appropriate facility listed below.

In the United States:

Clarity Service Center
4289 Bonny Oaks Drive, Suite 106
Chattanooga, Tennessee 37406
Tel. (423) 622-7793 or
(800) 426-3738
Fax: (423) 622-7646 or
(800) 325-8871

In Canada:

Plantronics Service Center
8112 Trans Canada Highway
Ville Saint-Laurent
Quebec H4S 1M5
Tel: (800) 540-8363
(514) 956-8363
Fax: (514) 956-1825

Please use the original container, or pack the unit(s) in a sturdy carton with sufficient packing material to prevent damage. Include the following information:

1. A proof-of-purchase indicating model number and date of purchase.
2. Bill-to address
3. Ship-to address
4. Number and description of units shipped
5. Name and telephone number of person to call, should contact be necessary
6. Reason for return and description of the problem. Damage occurring during shipment is deemed the responsibility of the carrier, and claims should be made directly with the carrier.

Clarity, a Division of Plantronics, Inc.
4289 Bonny Oaks Drive - Chattanooga, TN 37406
(800) 426-3738 www.clarityproducts.com

©2008 Clarity, a Division of Plantronics, Inc. All Rights Reserved. Clarity, the amplification device, Clarity and the amplification device combined, Amplifying Your Life, and Clarity Power are trademarks or registered trademarks of Plantronics, Inc.