clarity[®]

TL100 TV Listener

WHAT'S IN THE BOX?

TL100 headset

TL100 transmitter

Power adapter and
cord (2 sets)

RCA stereo Cable
(for option 1)

headphone cable
(for option 2)


Your new Clarity TL100 TV Listener is ready to provide high quality television sound. Setup is easy and should take just a couple of minutes.

*Clarity TL100 Headset is Plantronics BackBeat® SENSE, optimized for TV listening.


BASIC SETUP

STEP 1 Before connecting power to the transmitter, connect the transmitter to your TV, cable box, or satellite box, by choosing one of the two options below.

Option 1 (recommended, if available): Plug the red and white ends of the RCA stereo cable from the transmitter into the corresponding red and white **audio output** jacks on the back of your TV **or** on the back of your cable or satellite box. This allows you to adjust the headset volume while others can adjust the volume of the television speakers.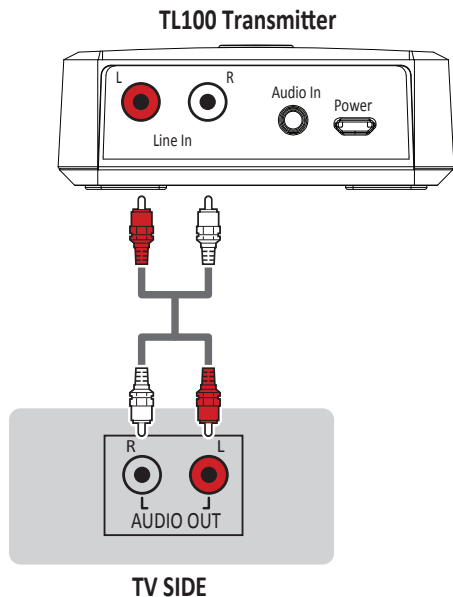

STEP 1 (continued)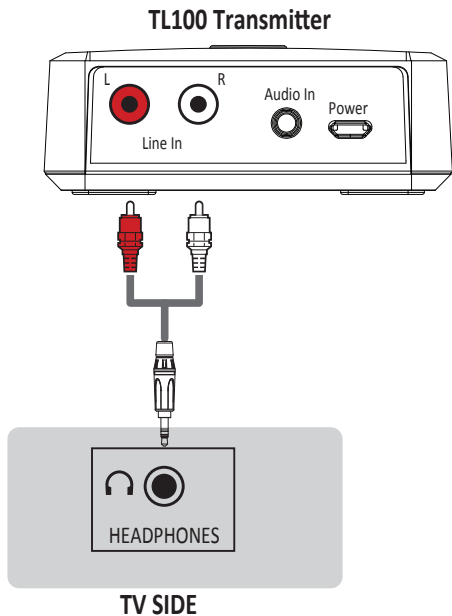

Option 2: Plug the headphone cable into the headphone jack on your TV. On most televisions, this disables the television's speakers, so this option is not ideal for multiple viewers. Check the settings on your TV for available functionality.

STEP 2 Plug one of the supplied power adapters into a power outlet, then plug the loose end of the power cable in the back of the transmitter. The light atop the transmitter will blink **red** three times to indicate that it has power, then starts blinking blue (searching for the headset).

STEP 3 Please make sure the headset is within range of the transmitter (up to 30ft). Turn on the headset by moving the small round switch (power switch) on the right side of the headset into center position (see page 6 to locate the power switch). The headset and transmitter are pre-paired and will connect automatically within 15 seconds. The transmitter light will turn solid blue when the headset is connected.

WHAT THE TRANSMITTER LIGHT MEANS:

Blinks red and turns off –transmitter off


Steady blue – headset connected

Blinking blue – searching for headset or headset out of range

Steady purple – headset off

USING YOUR HEADSET

Identify all controls of your headset according to the diagram below; see descriptions of functions on the next page (page 7).

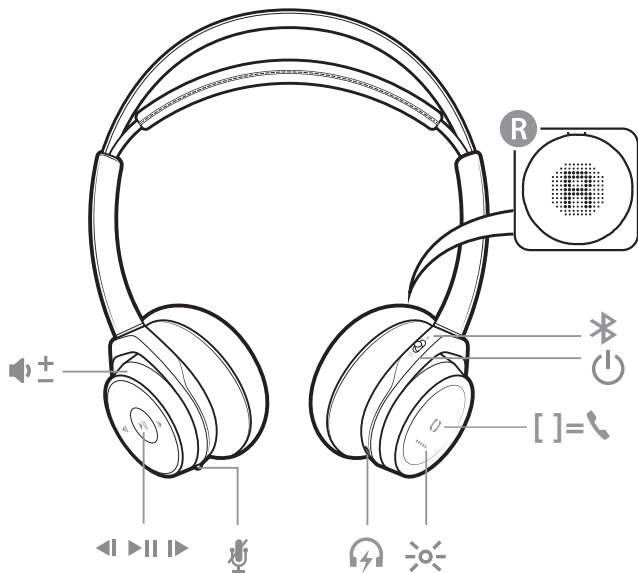HEADSET ADJUSTMENTS:

Toggle the Volume +/- wheel forward or backward to control the volume.

Tap the red Mute button to hear your natural surroundings. Tap the button again to

6 return to listening to your TV.

Adjust the headset to fit your head comfortably by pulling downward gently on each side, until the earphone cushions fit comfortably over each ear.

(controls for TV audio and music from mobile phone)

Volume up/down

Mute/unmute; Open mic

Charge port

Headset LEDs

Call button

Power switch

Bluetooth pair button

(controls for music from mobile phone only - see pairing instructions on page 8)

Track forward

Play/pause music

Track backward


PAIRING TO YOUR BLUETOOTH®-ENABLED MOBILE PHONE

1. Turn headset off, then turn back on; press the Call button once ([]).
2. On the right earpiece of the headset, slide and hold the power switch towards the Bluetooth icon until the LED's under the Call button ([]) flash red and blue.
3. Activate Bluetooth on your mobile phone and set it to search for new devices.

iPhone: Settings > Bluetooth > On

Android: Settings > Bluetooth: On > Scan for devices

NOTE: Menus may vary by device.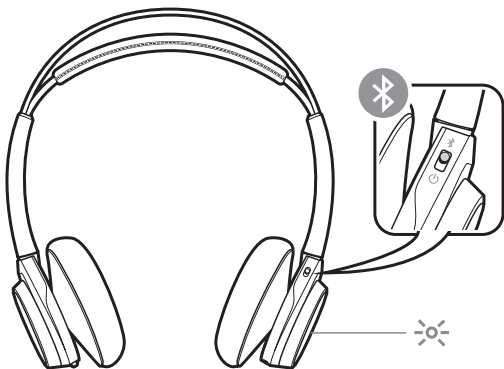

4. Select "Clarity TL100 Headset". If necessary, enter four zeroes (0000) for the passcode or accept the connection.

Once successfully paired, the LED's stop flashing.

5. Press the Call ([]) button to return to listening to the TV.

Make/Answer/End Calls:

1. To make a call: press the Call button ([]), then dial the number from the mobile phone.
2. To answer a call: you will hear an incoming call alert while listening to the TV. Press the Call button to answer. TV audio is muted during the call; streaming audio from the mobile phone is muted and paused.
3. To end a call: press the Call button to end the call. Headset returns to TV listener mode or resumes audio from mobile phone, respectively.
4. To decline a call: press and hold the Call button for 2 seconds; you will resume the previous audio activity from before the incoming call.

Mute: during a call, press the red Mute button. You hear "mute on/off".

LISTENING TO MUSIC (from mobile phone)

When you use your headset to listen to music from your mobile phone, the left headphone includes buttons for pause/play, next, and back (see pages 6 and 7).


To mute the streaming audio, tap the red Mute button. This allows you to hear your natural surroundings (audio streaming from mobile phone will pause). Press same button to resume streaming music/audio from mobile phone.

IMPORTANT: In order to preserve battery charge, we recommend turning off the headset when not in use.

CHARGING

Use the second power adapter and cable to charge your headset. It takes up to 2.5 hours to fully charge your headset. The LED's turn off once charging is complete.

To check the headset's battery status, tap the Call button [] while wearing your headset. Headset must be in idle state. Your headset have up to 18 hours of listening time.

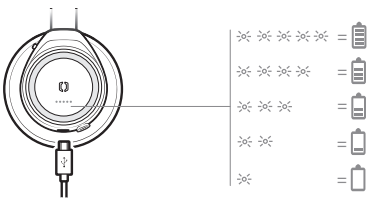TECHNICAL DETAILS

- TL100 HEADSET

- Weight: 140 grams
- Up to 18 hours of wireless listening time from rechargeable battery
- 5 LED battery gauge
- Up to 21 days standby time
- High quality audio
- Low latency performance
- Multipont technology

- TL100 TV LISTENER

- Range: up to 30 ft
- Answer mobile calls from headset (when Bluetooth-enabled mobile phone is paired to the headset)

en

Safety Instructions To reduce the risk of fire, electric shock, injury to persons and damage to property, read all operating instructions and the following safety instructions before using your Clarity product.

Operating, storage and charging temperature is 10°C to 40°C (50°F to 104°F).

- Only use those Clarity products and accessories designed for use with this product.
- If you experience a skin irritation after using this product, discontinue use and contact Clarity.
- CHILDREN. Never allow children to play with the product-small parts may be a choking hazard.
- Do not disassemble the product as this may expose you to dangerous voltages or other risks. Incorrect reassembly can cause electric shock or fire when the product is subsequently used.
- Unplug, discontinue use and contact Clarity if the product overheats or has a damaged outer casing, cord or plug.
- Exposure to high volume sound levels or excessive sound pressure may cause temporary or permanent damage to your hearing. Although there is no single volume setting that is appropriate for everyone, you should always use your headset/headphones with the volume set at moderate levels and avoid prolonged exposure to high volume sound levels. The louder the volume, the less time is required before your hearing could be affected. You may experience different sound levels when using your headset/headphones with different devices. The device you use and its settings affect the level of sound you hear. If you experience hearing discomfort, you should stop listening to the device through your headset/headphones and have your hearing checked by your doctor. To protect

your hearing, some hearing experts suggest that you:

1. Set the volume control in a low position before putting your headset/headphones on your ears and use as low a volume as possible.
2. Avoid turning up the volume to block out noisy surroundings. Whenever possible, use your headset/headphones in a quiet environment with low background noise.
3. Limit the amount of time you use headsets/headphones at high volume levels.
4. Turn the volume down if the sound from the headset/headphones prevent you from hearing people speaking near you.

See [plantronics.com/healthandsafety](https://www.plantronics.com/healthandsafety) for more information on headsets/headphones and hearing.

- Do not use headphones that cover both ears while driving or riding a bicycle. Such use is unlawful in many regions and will impair your ability to hear sounds around you, creating a hazard for you and others.

Charging Warnings

- Use only the charger supplied by Clarity to charge the product and follow the charging instructions provided. Do not use the charger for any other purpose. Ensure that the voltage rating corresponds to the power supply you intend to use.

Battery Warnings

- This product has an embedded, non-replaceable battery; do not attempt to open the product or remove the battery as this may cause injury and/or damage the product.
- Do not dispose of the product in a fire or heat above 40°C (104°F).
- RECYCLING The product must be recycled or disposed of properly. Contact your local recycling center for battery removal and proper disposal.

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Registering Your Product

Visit http://shop.clarityproducts.com/support/warranty_registration to register your product online so we can provide you with the best service and technical support.

Limited Warranty

Clarity offers a one (1) year limited warranty. For warranty details, conditions and technical assistance, please visit our website: www.clarityproducts.com/warranty.

Clarity

6131 Preservation Dr.
Chattanooga, TN 37416
United States

©2016 Plantronics, Inc. Clarity and Plantronics are trademarks of Plantronics, Inc. registered in the US and other countries. TL100 is a trademark of Plantronics, Inc. Bluetooth is a registered trademark of Bluetooth SIG, Inc. and any use by Plantronics, Inc. is under license. iPhone is a trademark of Apple Inc. registered in the US and other countries. All other trademarks are the property of their respective owners.

(RevA - 09.15.2016 pn4952.001)

NEED MORE HELP? CALL 800-426-3738