




PACKAGE CHECKLIST

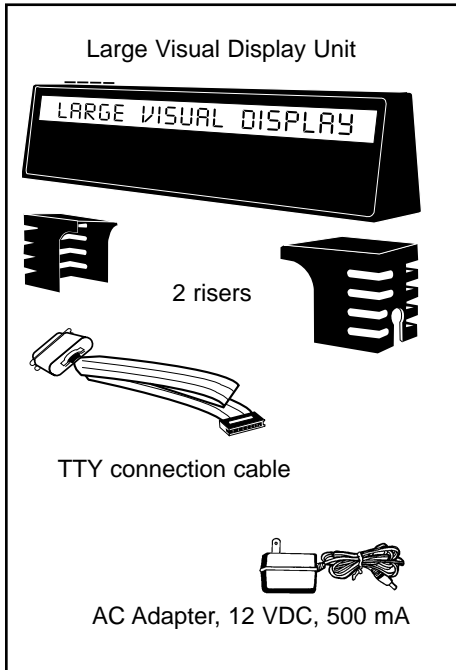

CONNECTING THE LVD

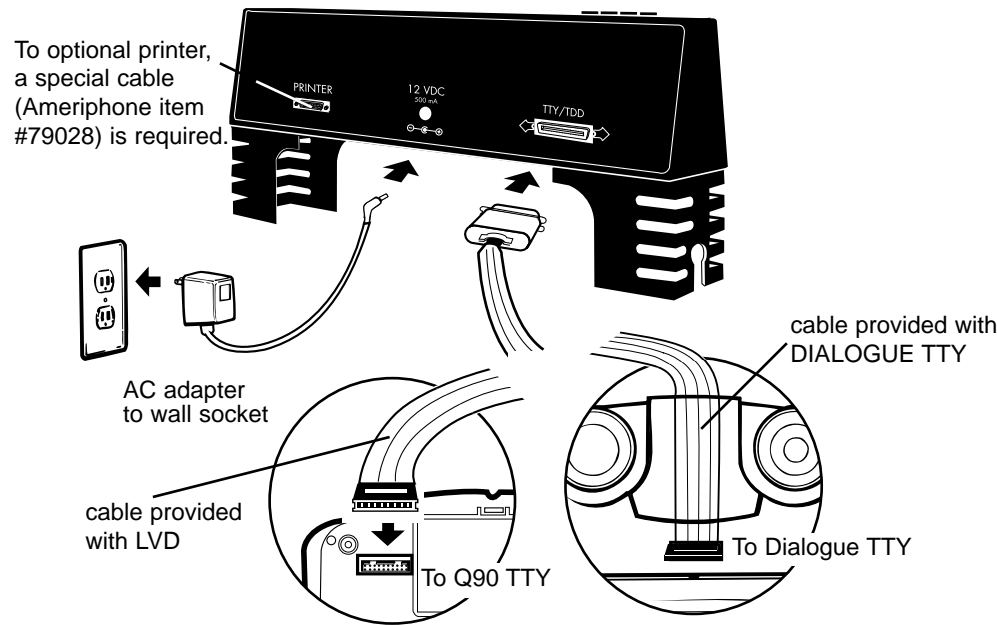

FUNCTIONS

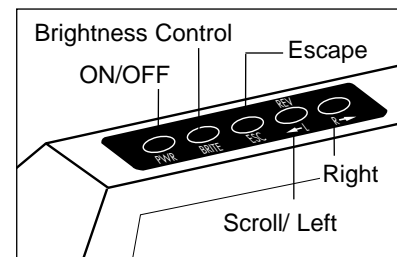

TO ERASE MESSAGES

To erase all previous messages from the display panel, make sure the LVD is OFF.

- 1 Press and hold the Escape key,
- 2 Then press the On/ Off button and release
- 3 Then release the Escape key

Please note that the LVD's conversation memory will be retained for up to two weeks, even if the unit is unplugged.

Scrolling through your messages

- 1 To enter Scroll mode, first press the Scroll/ Left button
- 2 Then, you can press either the Left or the Right button to scroll in the direction you desire to see your display panel messages.
- 3 You can increase the scrolling speed by holding down the Left or Right button while in Scroll mode.
- 4 Press either the Escape or the Right key to exit the Scroll mode and return to where you started.

When you are in Scroll Mode, lights will appear over the letters on the display panel.



IN CASE OF DIFFICULTY

Symptom	Cause and Remedy
Won't come on.	Unit not plugged in. Check connection plug.
Panel display is blank	Make sure LVD is On. Make sure the TTY is On.

SPECIFICATIONS

SIZE: WIDTH – 18.9"
DEPTH – 3.1"
HEIGHT – 4.4"

DISPLAY COLOR: Red. Green and Amber are also available

WEIGHT: 3 lbs.

POWER: 12 VDC, 500 mA AC Adapter

UL / CSA LISTED ADAPTER

FCC STATEMENT

This product complies with FCC Part 15 requirements.

WARRANTY

Ameriphone warrants the Large Visual Display against any defect in materials or workmanship for the period of one year from the date of purchase. If our extended warranty is purchased, this same warranty also applies to the period covered by the extended warranty.

If your Ameriphone product is defective and returned within 30 days of the date of purchase, your Dialogue Large Visual Display dealer will replace it at no charge.

If returned after 30 days but within one year from the date of purchase, we will repair or replace it at no charge. In the repair of your LVD, we may use new or reconditioned replacement parts. If we elect to replace your LVD, we may replace it with a new or reconditioned product of the same or similar design. Repair or replacement will be warranted for either 90 days or the

WARRANTY con't

remaining time on the original warranty period, whichever is longer. When in need of warranty repair, please call our customer service department at (800) 874-3005 (voice), (800) 772-2889 TTY. They will help you process your return shipment.

Implied warranties, including those of fitness for a particular purpose and merchantability (an unwritten warranty that the product is fit for ordinary use), are limited to one year from date of purchase. We will not pay for loss of time, inconvenience, loss of use of your Dialogue LVD, or property damage caused by your Dialogue LVD or its failure to work, or any other incidental or consequential damages. Some states do not allow limitations on how long an implied warranty lasts or the exclusion of incidental or consequential damages, so the above exclusions or limitations may not apply to you.

To get warranty service for your Dialogue LVD, you must provide proof of the purchase date. Within 30 days of the date of purchase, return your LVD to the

place where you purchased it for immediate replacement. After 30 days, call Ameriphone at (714) 897-0808 (voice), (714) 897-1111 (TTY/TDD) for the authorized service center nearest you. We suggest you save the original package materials in the event you need to ship the LVD. You must prepay all shipping costs. When shipping for warranty repair, include your name, address, phone number, proof of date of purchase and a description of the problem. After repairing the product, we will return it back to you at no cost within the United States.

This warranty does not cover defects resulting from accidents, damage while in transit to our service location, alterations, unauthorized repair, failure to follow instructions, misuse, use outside the United States or Canada, fire, flood and acts of God. If your LVD is not covered by this warranty, call us at (714) 897-0808 (voice)/(714) 897-1111 (TTY/TDD) for advice as to whether we will repair your LVD and other repair information. This non-covered repair shall be warranted for 90 days.

Patent Pending
Precision Engineered in the U.S.A.
Assembled in China

©AMERIPHONE, 5290-4005 ▲9100 M00800

DIALOGUE LVD

OPERATING INSTRUCTIONS



AMERIPHONE, Inc., 12082 Western Avenue
Garden Grove, CA 92841
(714) 897-0808 Voice (714) 897-1111 TTY
(714) 897-4703 Fax
ameriphone.com www.ameriphone.com